

DENTAL HISTORY AND DIAGNOSIS

White Spots

Fluorosis

Trauma

MIH

Diagnosis depth of the spot

Superficial lesion

Mixed lesion

Deep lesion

PRE-TREATMENT

Bleaching (if necessary)

Waiting for 2 weeks



Rubber Dam/ Liquid Dam

Microabrasion (optional)

Sandblasting/Microabrasion

Diamond Bur

INFILTRATION

Icon-Etch



Repeat

Spots diminished

Icon-Dry



Spots are still visible

Icon-Infiltrant



Light-curing



Light-curing



Icon-Infiltrant



Add Composite (optional)

Polishing