

G-Premio BOND Technique Guide



Select etching mode



Self-etch



10-15
SEC.

Selective etch

Etch enamel only
for 10-15 sec., rinse
for 5 sec. and dry.



10-15
SEC.

Total-etch

Etch for 10-15
sec., rinse for
5 sec. and dry.

Bonding procedure



10
SEC.

1. Apply G-Premio BOND and
wait 10 sec.



5
SEC.

2. Dry for 5 sec. (MAX air pressure).



10
SEC.

3. Light-cure for 10 sec.



4. Start placing composite

G-Premio BOND Technique Guide



Intraoral repair



1. Roughen surface. Rinse and dry.



2. For all glass ceramics and hybrid ceramics, apply G-Multi PRIMER. Dry.



3. Apply G-Premio BOND to all surfaces and wait 10 sec.



4. Dry for 5 sec. (MAX air pressure).



5. Light-cure for 10 sec. Begin composite application to complete repair.



G-Multi PRIMER



G-Premio BOND

These products are not available for purchase by the general public. Always read the label and follow the directions for use.

GC Australasia Dental Pty Ltd
1753 Botany Road Banksmeadow
NSW 2019 Australia

T: +61 2 9301 8200
E: info.australasia@gc.dental
www.gcaustralasia.com