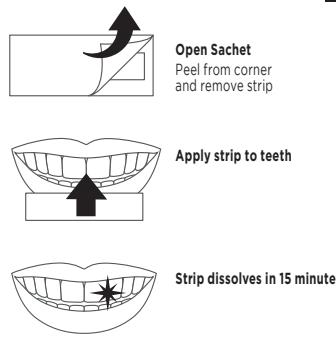
PATIENT INSTRUCTIONS

Instruction for use:

1. Use strip immediately after opening foil sachet
2. Remove the strip and apply either side of the strip to upper teeth.
3. Align the edge of the strip with the gum line and press the strip in place.
4. Repeat for lower teeth.
5. The strips will dissolve gradually leaving no residue.
6. Use once or twice daily as required

Recommended Wear Time:

The whitening strip will dissolve in around 15 minutes

Precautions

1. Pregnant and breastfeeding women should consult their physician before using whitening strips.
2. Keep out of direct heat/sunlight. Do not freeze.
3. Tooth fillings and crowns do not respond to tooth bleaching
4. Do not use tobacco products while bleaching
5. If you have any questions regarding the appropriate use of this product, please consult your dentist as soon as possible.
6. Contains Hydrogen Peroxide

Storage temperature 5 - 25°C

EN.1

UK RP: MDSS Ltd. 6 Wilmslow Road, Rusholme, Manchester M14 5TP, UK
EU RP: MDSS GmbH - Schiffgraben 41, 30175 Hanover, Germany

PROFESSIONAL DISSOLVABLE WHITENING STRIP

INSTRUCTIONS FOR USE

DISTRIBUTED BY:
Opident Ltd
Valley Drive, Ilkley, West Yorkshire, LS29 8AL
Tel: +44 (0) 1943 605 050
Email: sales@opident.co.uk
Website: https://www.opident.co.uk

MADE IN UK
REF: WDBSTRIPUK
S1 REV: 09/22

PL: System wybielania z bów
NO: Tannblekningssystem
FR: Système de blanchiment des dents
NL: Tandbleekingsysteem
IT: Sistema di sbiancamento dei denti
ES: Sistema de blanqueamiento dental
AR: ناس الـ سـيـنـيـهـ بـتـ

Warnings


- To be only sold to dental practitioners.
For each cycle of use, the first use to be only done by dental practitioners or under their direct supervision if an equivalent level of safety is ensured. Afterwards to be provided to the consumer to complete the cycle of use
Products containing peroxides are not recommended for use by children under 18 years of age.
Warnings: If irritation (such as redness, swelling, soreness) of the gums or the mouth occurs, discontinue use and consult a dentist. Use of product for periods of longer than 14 days is to be only under the supervision of a dentist.
Avoid swallowing the strip or part thereof.
Avoid contact of the product with the eye.
Avoid direct contact of the active surface of the tooth whitening product with the gums and/or salivary flow.

EN.4

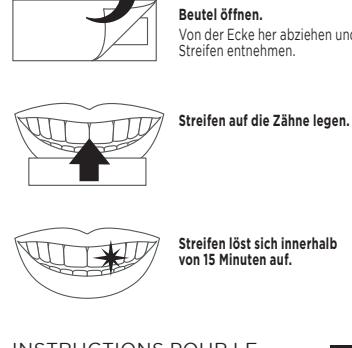WARNING
H315: Causes skin irritation.
H317: May cause an allergic skin reaction.
H318: Causes serious eye damage.
H226: Flammable liquid and vapour.
H400: Very toxic to aquatic life.
H411: Toxic to aquatic life with long lasting effects.

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