

Date 12/09/2023

Acteon Wrenches

To prepare your medical device, follow the steps below:

Wear safety goggles and protective gloves.

Clean the unit with an alcohol disinfectant wipe.

Remove the handpiece from its sterilization bag.

Remove the wrench from its sterilization bag.

Remove the tip from its sterilization bag.

Screw the tip onto the handpiece, first manually and finishing with the wrench.

Connect the handpiece to the handpiece cord socket.

Place the handpiece on its support.

Switch on the medical device.

Check the irrigation parameters depending on the tip chosen.


After water drainage, check that the spray works correctly.

Your medical device is now ready to use 1.

Place the handpiece on the support.

A tip or a file vibrates correctly when it is perfectly tightened without being forced beyond its stop point. Tighten the tip with the torque wrench (F81320, F81321, F81322 or F81323) to ensure optimum ultrasonic function. Over-tightening of the tip or file with the open-ended wrench can result in breakage of the tip, file or handpiece.

| To prevent self-locking of the tip or the file, the latter must be removed and sterilised after each use.

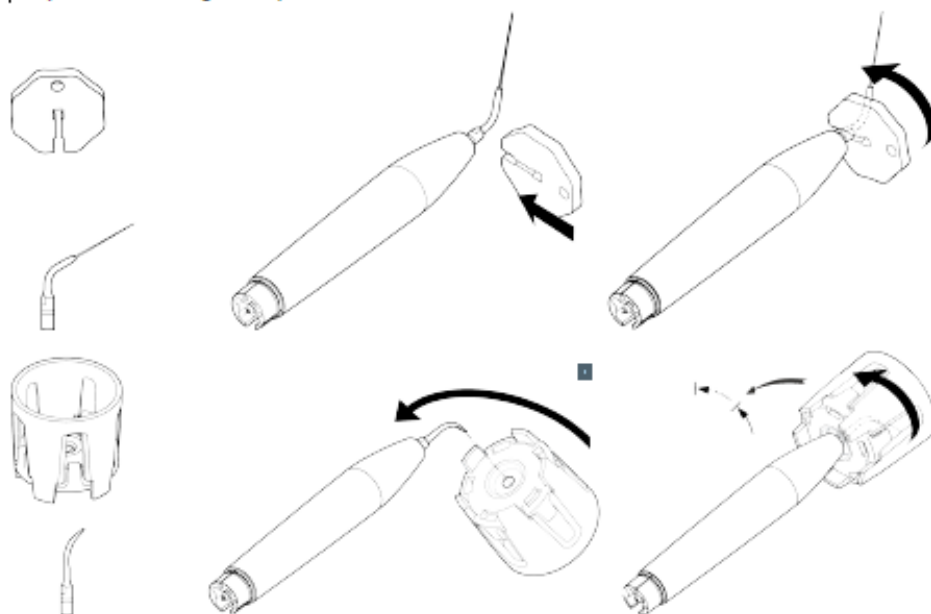

ACTEON Australia / New Zealand | Suite 1b, 13B Narabang Way | Belrose | NSW 2085 | Australia
Tel. +61 (0) 2 9669 2292 | Free call 1300 ACTEON (1300 22 83 66) | Fax: +61 (0) 2 8569 1739